


DuraSoft[®] Laser Printable Patient ID System Instructions

Step 1:

Print patient ID information and bar codes onto the DuraSoft sheet and remove it from your laser printer.

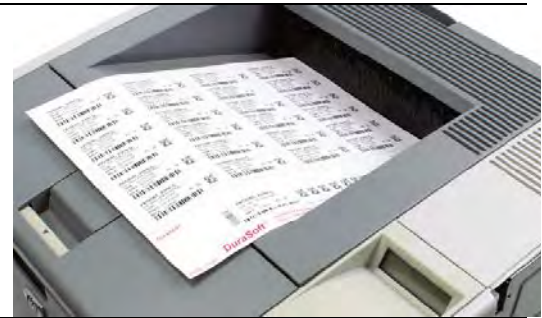


Step 2:

As indicated on the DuraSoft sheet, peel the wristband by starting at the adhesive tab.



Step 3:

As indicated, hold and tear at the perforation to remove the wristband from the sheet.

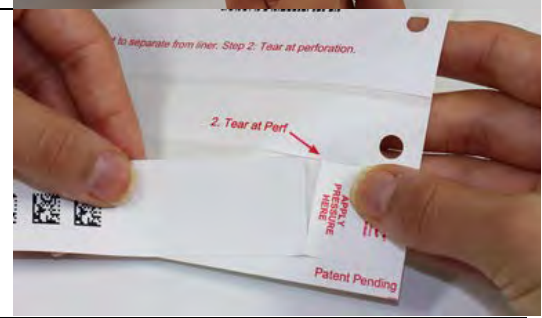

Step 4:

Apply wristband to patient's wrist and secure with adhesive closure. Be sure to leave enough room to ensure comfort and a secure fit.

